HICUT INSTRUMENTS

EN INSTRUCTIONS FOR USE

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EN

General

adeor HiCUT instruments are manufactured from high quality materials. In order to achieve best possible results, the following instructions for use must be followed.

Misuse can result in destruction of the instrument and risk of injury to the patient and/or user.

All HiCUT instruments are single-use products. Re-using the instruments is not allowed.

Intended use

HiCUT instruments are intended for trephination, incision, cutting, removal, shaping, sawing of soft and hard tissue, bone and biomaterials.

Instruments for third party systems are to be used with the corresponding systems only and according to the instruction manual of the corresponding third party system.

adeor Velocity Alpha cutters, burs and saws are to be used with the corresponding adeor Velocity Alpha system components only.

The single-use packaging must be undamaged and the red dot sticker (indicator for sterile product) attached to the packaging. If the sterile packaging is damaged or the red dot sticker neither red nor visible, the product must not be used.

Special reference to saw-blades

Please ensure that the saw blades are inserted into the corresponding attachment and into the deepest possible position. Unclamped or exposed saw blades cause high centrifugal force and vibrations.

The maximum saw speed is 40.000 rpm.

This speed must not be overstepped. Misuse can result in destruction of the instrument and risk of injury to the patient and/or user.

Controlled pressure

Activate the instruments before positioning them on the bone structure. During operation, refrain from tilting, levering or bending (risk of breakage).

Extensive contact pressure must be avoided at all times. High contact pressure may lead to thermal necrosis. In extreme cases, instrument failure may occur.

Irrigation

To avoid unwanted overheating, ensure sufficient irrigation. Non discharged material may lead to decreased cutting performance. In this case the instruments cannot work efficiently which may lead to increased temperature.

Storage

Instruments must be kept dry, protected from light and stored at room temperature. Do not use if the stated expiration date has passed.

Disposal

Legal regulations must be followed when disposing HiCUT instruments.

Safety and liability

adeor medical AG products are designed and manufactured with great care. The user is obligated to self dependently examine the applicability of the product to the intended application. The user bears the responsibility for the application of the instruments. Resulting damages are excluded from liability.

Symbols

-  Follow instruction for use
-  Do not use if damaged
-  Use by - symbol is accompanied by a date
-  Do not reuse / single-use only
-  Sterile by radiation
-  Catalogue no.
-  Lot no.
-  Prescription device only (USA)

adeor medical AG
Martinshof 5
83626 Valley
Germany

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